# PMW CONTROL SWAB

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## 1. Manufacture

PMW Corp.

- Headquarters : Room 904, 48, Centum jungang-ro, Busan, Republic of Korea
- Side Address : Room A, 2F, 135, Cheoyongsaneop 3-gil, Cheongnyang-eup, Ulju-gun, Ulsan, Republic of Korea

2. Registration No. : No. 21-603

3. Product Name : PMW-SWAB

4. Model Name : PMW CONTROL SWAB | PMW CONTROL SWAB L15 | PMW CONTROL SWAB-B

## 5. Product Composition

Model Name	Components
PMW CONTROL SWAB	1 EA
PMW CONTROL SWAB-B	1 EA
PMW CONTROL SWAB L15	1 EA

## 6. Purpose of usage

PMW CONTROL SWAB and PMW CONTROL SWAB L15 are a method for collecting a clinical test sample from nasal or oral cavity. The sample is then analyzed for the presence of organisms or other clinical markers for disease.

## 7. How to Use

- ① Preparation before usage
  - Check for any damages on the package and product before usage.
- ② How to use
  - 1) Open the product
  - 2) Carefully take the product out of the package.
  - 3) Take specimen from the patient.
  - 4) Insert the head section of the product in a separate preservation container or transportation container and exert force at the break point.
- ③ How to store and maintain the product after usage
  - 1) It is disposable product. Discard after usage.
  - 2) Do not reuse.

## 8. Precautions for use

- ① Warning
  - 1) The product must be used by medical professionals
  - 2) It is a disposable product. Do not reuse the product
  - 3) Do not use the damaged product
- ② Precautions for use
  - 1) Check for any damages to the product before usage.
  - 2) Be careful not to exert excessive force when collecting specimens, such as putting forced pressure.
  - 3) Avoid direct sunlight. Store at room temperature.