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PMW POLYESTER SPUN SWAB & PMW RAYON SWAB

1. Manufacture

PMW Corp.

- Headquarters : Room 904, 48, Centum jungang-ro, Busan, Republic of Korea
- Side Address : Room A, 2F, 135, Cheoyongsaneop 3-gil, Cheongnyang-eup, Ulju-gun, Ulsan, Republic of Korea

2. Registration No. : No. 20-1963

3. Product Name : PMW Spun Swab

4. Model Name : PMW POLYESTER SPUN SWAB / PMW RAYON SWAB

5. Product Composition

Model Name	Components
PMW POLYESTER SPUN SWAB	1 EA
PMW RAYON SWAB	1 EA

6. Purpose of usage

PMW POLYESTER SPUN SWAB and PMW RAYON SWAB are a method for collecting a clinical test sample from nasal or oral cavity. The sample is then analyzed for the presence of organisms or other clinical markers for disease.

7. How to Use

① Preparation before usage

- Check for any damages on the package and product before usage.

② How to use

- 1) Open the product
- 2) Carefully take the product out of the package.
- 3) Take specimen from the patient.
- 4) Insert the head section of the product in a separate preservation container or transportation container and exert force at the swab's body.

③ How to store and maintain the product after usage

- 1) It is disposable product. Discard after usage.
- 2) Do not reuse.

8. Precautions for use

① Warning

- 1) The product must be used by medical professionals
- 2) It is a disposable product. Do not reuse the product
- 3) Do not use the damaged product
- 4) Do not use the product vertically

② Precautions for use

- 1) Check for any damages to the product before usage.
- 2) Be careful not to exert excessive force when collecting specimens, such as putting forced pressure.
- 3) Avoid direct sunlight. Store at room temperature.
- 4) Be careful when collecting specimens.